USER MANUAL

Anti-bedsore cushion

Product code: **BUBBLE EVERYDAY,**
BUBBLE EXPERT, BUBBLE EXCLUSIVE

1. Characteristics

Cushion is made of high quality materials. Individual air cells connected into one air system ensure adequate body balance. Pressure exerted to one air cell causes an increase of pressure in adjacent air cells, which allows particular body parts to be relieved. Multi air cells design and the possibility to adjust hardness by inflating or releasing air provide high comfort of use. In addition BUBBLE EXPERT is equipped with an extra valve which allows to adjust cushion's hardness, depending on individual preferences. Cushions BUBBLE EVERYDAY & BUBBLE EXCLUSIVE have an efficient manual pump.

The cushion is manufactured using heat sealing method to ensure its durability and tightness.

It has a protective cover with a zip fastener, made of a mesh material with an anti-slip bottom. A handy holder facilitates manual carrying. The protective cover has additional Velcro fasteners that allow the cushion to be attached if required. The kit contains: cushion, protective cover, manual pump (BUBBLE EVERYDAY and BUBBLE EXCLUSIVE).

2. Application and recommendations

The pneumatic anti-bedsore cushion is an auxiliary device intended for patients exposed to the risk of bedsores. It is used by persons remaining in sitting position for a prolonged period of time with circulatory disorders in systemic cardiovascular system. It is also used in bedsore preventive care and in treatment of bedsores up to the first degree (acc. to NPUAP/EPUAP).

Avoid contact with bare skin. Before use consult with your doctor, rehabilitator or physical therapist.

3. Notes and recommendations

Timago International Group does not bear responsibility of improper use of the product, failure to comply safety regulations and misuse.

4. Method of use

- Unfold the cushion on a flat surface.
- Open the air valve and insert the pump (in BUBBLE EVERYDAY and BUBBLE EXCLUSIVE) or turn the valve anticlockwise – the cushion will be inflated automatically (BUBBLE EXPERT).
- Pump to the required hardness.
- Fit the protective cover.

To release air:

- In BUBBLE EVERYDAY and BUBBLE EXCLUSIVE – open the air valve and squeeze it with your index finger and thumb.
- In BUBBLE EXPERT – turn the valve anticlockwise, sit on the cushion and leave the valve open.

5. Technical data

Model	BUBBLE EVERYDAY	BUBBLE EXPERT	BUBBLE EXCLUSIVE
Dimensions	42 x 42 x 4 cm	46 x 38 x 4 cm	41 x 46 x 7 cm
Maximum load	120 kg	120 kg	143 kg
Weight	0,65 kg	0,65 kg	0,62 kg
Cover	breathable, zipper	breathable, zipper	breathable, zipper
Color	black	black	black

6. Comments and remarks

- Pump the cushion using the supplied pump only (for BUBBLE EVERYDAY and BUBBLE EXCLUSIVE). Make sure to not damage the air valve during pumping.
- Do not pump with a compressor. Excessive pressure may tear the cushion apart.
- The BUBBLE EXPERT is provided with an automatic valve, so no pumps should be used (the cushion is inflated automatically).
- Do not exceed the maximum cushion load.
- Do not stand on the cushion.

- Before using the cushion on a light upholstery fabric carry out a test as there is a probability of stamping dots from the anti-slip bottom of the cover on the fabric.
- Store in a dry room, away from fire and sharp objects.
- Do not expose to high temperatures and sunlight as the increase of temperature inside the cushion may lead to its bursting.

7. Maintenance

Cleaning:

- Clean the cushion with a damp cloth moistened with mild cleaning agents. Do not wet the cushion. Wipe it dry after every maintenance. Strong cleaning agents or detergents may damage the product. Do not use abrasives.
- **Attention!** Before cleaning close the pumping valve to protect it from wetting.
- **Attention!** Do not use heat sources for drying.
- **Attention!** Do not use whitening agents, fabric softeners and dry cleaning agents for cleaning!

Protective cover cleaning:

- The protective cover can be washed manually or in the washing machine at 40°C using mild detergents. Before cleaning fasten the zip. The protective cover can be dried with a stream of warm air or in a drier at a low temperature.
- **Attention!** The cushion can be fitted with the protective cover only when the latter has completely dried.
- Do not iron the protective cover!

8. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.

9. Warranty information

All products distributed by our company are under a warranty, the terms of which are described in the warranty card received at the time of purchase.

Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

10. Labels

	Reference number		Manufacturer
	LOT number		Manufacture date
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors.
Learn more at: www.timago.com

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