## KING RAISED TOILET SEATS

WITH LATERAL LOCKING SYSTEM

EN USER MANUAL  
DE BENUTZERHANDBUCH  
FR MANUEL DE L'UTILISATEUR  
ES MANUAL DE USUARIO  
IT MANUALE D'USO  
PT MANUAL DO USUÁRIO  
SL NAVODILA ZA UPORABO  
HR KORISNIČKI PRIRUČNIK  
HU HASZNÁLATI UTASÍTÁS  
RO MANUAL DE UTILIZARE

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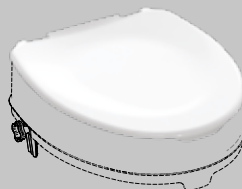
## WITH LATERAL LOCKING SYSTEM

### INTENDED USAGE:

The products of the *Raised Toilet Seat with Lateral Locking System KING* line are designed to increase the height of the toilet bowl to facilitate sitting down and getting up procedures and promote independence and comfort. The models provided with soft cushion offer improved seat comfort, making them soft and cozy. In addition, the wrinkled surface of the cushion provides more friction and stability. The product comes in different configurations, with or without a soft cushion, and with or without a lid. The soft cushions are available in different colors. The lid, bidet, and soft cushions are also available separately as accessories.

### INDICATIONS AND CONTRAINDICATIONS:

The products of the Raised Toilet Seat with Lateral Locking System KING line are aimed at elderly people, wheelchair users, people with reduced mobility of the lumbar, knee, or hip, and to all those who might have difficulty in sitting down and getting up from the toilet bowl. The models provided with soft cushion deforms during sitting, increasing the user support surface and allowing for better weight distribution. This makes these versions particularly suitable if the user suffers from bedsores or diseases which could cause them, such as diabetes. The ability to sit independently is required during use. Using the product in case of perception disorders or balance problems while sitting is not recommended.

### SAFETY WARNING AND INDICATIONS:

- The maximum user weight indicated in the fitting instructions must not be exceeded.
- Before every use, be sure the product is in a stable position on the WC and correctly fixed on it by the locking system supplied.
- Be sure the soft cushion wraps the seat correctly.
- Before applying the soft cushion, be sure the seat is clean and dry.
- During the assembly, strictly adhere to what is shown in the User Manual.
- Do not modify the product and its components in any way.
- Do not use the product if damaged or altered in any way.
- Be sure the product height is such as making it easy and safe for the user.
- During use, avoid positions that keep the body weight too much forward. Wrong positions may lead to instability and overturning of the product.
- The product and the accessories must not be stored outdoors.

### MAINTENANCE INFO:

Product maintenance consists of cleaning and sanitation procedures. In order to guarantee safe use and an adequate standard of hygiene, the user should perform these procedures before every use. The user has to make certain of the structural integrity of the device and its components, and the tightness of the screws must be checked. For cleaning and sanitation procedures, follow the steps below:

1. Gloves should be worn.
2. Remove the evident dirt before proceeding with the most accurate disinfection.
3. Use water or non-aggressive detergent for the cleaning procedure.
4. Wipe with a dry and clean cloth.

Alternatively, the product can be cleaned in domestic dishwashers at the maximum temperature. The soft cushion must be cleaned strictly by hand.

### REUSE REQUIREMENTS:

The product is suitable for reuse by other users. Before being reused by others, the product must be correctly cleaned and sanitized. Before the transfer of the product to others, the product must be checked by qualified dealers.

### CORRECT DISPOSAL PROCEDURE:

Do not dispose of the product in the regular household waste. Ask your local authority beforehand for the correct and environmentally friendly way to dispose of it.

### INCIDENT REPORTING:

If there is a serious incident involving this device, contact the manufacturer of this product at [info@kingbathrooms.com](mailto:info@kingbathrooms.com) and the responsible authority in your country.

ITEM CODE	Description	PAGE
<b>KING-5-KN</b>	Raised toilet seat 5 cm/2in King	<b>44</b>
<b>KING-5L-KN</b>	Raised toilet seat 5 cm/2in with lid King	<b>44</b>
<b>KING-10-KN</b>	Raised toilet seat 10 cm/4in King	<b>44</b>
<b>KING-10L-KN</b>	Raised toilet seat 10 cm/4in with lid King	<b>44</b>
<b>KING-15-KN</b>	Raised toilet seat 15 cm/6in King	<b>44</b>
<b>KING-15L-KN</b>	Raised toilet seat 15 cm/6in with lid King	<b>44</b>
<b>KING-5S02/04-KN</b>	Soft raised toilet seat 5 cm/2in light blue/grey King	<b>45</b>
<b>KING-5LS02/04-KN</b>	Soft raised toilet seat 5 cm/2in with lid light blue/grey King	<b>45</b>
<b>KING-10S02/04-KN</b>	Soft raised toilet seat 10 cm/4in light blue/grey King	<b>45</b>
<b>KING-10LS02/04-KN</b>	Soft raised toilet seat 10 cm/4in with lid light blue/grey King	<b>45</b>
<b>KING-15S02/04-KN</b>	Soft raised toilet seat 15 cm/6in light blue/grey King	<b>45</b>
<b>KING-15LS02/04-KN</b>	Soft raised toilet seat 15 cm/6in with lid light blue/grey King	<b>45</b>
<b>KING-L-KN</b>	Lid for raised toilet seat 5/10/15 cm King	<b>46</b>
<b>KING-LS-KN</b>	Lid for soft raised toilet seat 5/10/15 cm King	<b>46</b>
<b>KING-BDT-KN</b>	Bidet for raised toilet seat 5/5L/10/10L/15/15L King	<b>47</b>
<b>KING-CRTS2/4-KN</b>	Soft cushion for raised toilet seat light blue/grey King	<b>47</b>

ICON	DESCRIPTION
	Made in Italy
	Tools Free Assembly
	Dishwasher Friendly
	Max User Weight
	2 Years Warranty
	Scan the QR CODE to watch the assembly video

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LM Global Design Ltd.