KING BATHROOM STOOLS AND ACCESSORIES

EN USER MANUAL
DE BENUTZERHANDBUCH
FR MANUEL DE L'UTILISATEUR
ES MANUAL DE USUARIO
IT MANUALE D'USO
PT MANUAL DO USUÁRIO
SL NAVODILA ZA UPORABO
HR KORISNIČKI PRIRUČNIK
HU HASZNÁLATI UTASÍTÁS
RO MANUAL DE UTILIZARE

UK ПОСІБНИК КОРИСТУВАЧА
SK ROUŽIVATEĽSKÁ PRÍRUČKA
CS UŽIVATELSKÝ MANUÁL
PL INSTRUKCJA OBSŁUGI
LT VARTOTOJO VADOVAS
ET KASUTUSUJUHEND
LV LIETOTĀJA ROKASGRĀMATA
FI OHJEKIRJA
SV ANVÄNDARMANUAL
NL HANDLEIDING

- KING-ST-KN
- KING-STAK-KN
- KING-BST-KN
- KING-BSTA-KN

- KING-STR-KN
- KING-STRA-KN
- KING-BSTR-KN
- KING-BSTRA-KN

- KING-STT-KN
- KING-STTA-KN
- KING-BSTT-KN
- KING-BSTTA-KN

- KING-STRR-KN
- KING-STTRA-KN
- KING-BSTRR-KN
- KING-BSTTRA-KN

- KING-STAT-KN
- KING-BSTAT-KN

- KING-STRAT-KN
- KING-BSTRAT-KN

- KING-STTAT-KN
- KING-BSTTAT-KN

- KING-STRRAT-KN
- KING-BSTRRAT-KN

KING-CST2-KN
KING-CST4-KN

KING-CSTT2-KN
KING-CSTT4-KN

KING-CSTR2-KN
KING-CSTR4-KN

KING-CSTRR2-KN
KING-CSTRR4-KN

- KING-RD-KN
- KING-BRD-KN

EN	USER MANUAL	4-5
DE	BENUTZERHANDBUCH	6-7
FR	MANUEL DE L'UTILISATEUR	8-9
ES	MANUAL DE USUARIO	10-11
IT	MANUALE D'USO	12-13
PT	MANUAL DO USUÁRIO	14-15
SL	NAVODILA ZA UPORABO	16-17
HR	KORISNIČKI PRIRUČNIK	18-19
HU	HASZNÁLATI UTASÍTÁS	20-21
RO	MANUAL DE UTILIZARE	22-23
UK	ПОСІБНИК КОРИСТУВАЧА	24-25
SK	POUŽÍVATEĽSKÁ PRÍRUČKA	26-27
CS	UŽIVATELSKÝ MANUÁL	28-29
PL	INSTRUKCJA OBSŁUGI	30-31
LT	VARTOTOJO VADOVAS	31-33
ET	KASUTUSJUHEND	34-35
LV	LIETOTĀJA ROKASGRĀMATA	36-37
FI	OHJEKIRJA	38-39
SV	ANVÄNDARMANUAL	40-41
NL	HANDLEIDING	42-43

KING BATHROOM STOOLS

FIXED OR HEIGHT ADJUSTABLE

EN

INTENDED USAGE:

The products of the *Stools KING* line allow increased independence during the shower stage of daily personal hygiene, reducing muscle fatigue due to standing position as well as slip and injury risks. The versions provided with non-slip rubber feet ensure better grip on the floor, whereas the models with height adjustable legs allow users to change the total height according to their needs easily. The versions provided with a rotating disc increase the freedom of movement without the need to stand up from the stool. Due to its flexibility, the stools can be positioned even on a bumpy and irregular floor, like those typical of many shower trays. The stools are available in three different shapes. The stools can be easily moved due to their lightweight. The stools can be completed with a soft cushion to make the use more comfortable. The soft cushions and rotating discs are available separately as accessories.

INDICATIONS AND CONTRAINDICATIONS:

The products of the Stools KING line are aimed at elderly people, wheelchair users, people with reduced mobility of the lumbar, knee, or hip, and all those who have difficulty to maintain the standing position during the shower stages of daily personal hygiene. The user is required to have autonomy of balance during sitting and standing procedures as well as to have adequate ability to keep sitting. The shower seat must not be used if the user has problems related to balance or perception disorders.

SAFETY WARNING AND INDICATIONS:

- The maximum user weight indicated in the fitting instructions must not be exceeded.
- Before every use, be sure the shower seat is correctly assembled in a stable position.
- The shower seat can be used only on a surface free from obstacles.
- During assembly, strictly adhere to what is shown in the User Manual.
- The risk of finger injuries may increase during the mounting process.
- Do not swing on the shower seat; this may lead to slipping, overturning, or subsidence of the legs. Always keep the shower seat on its 4 legs.
- Do not modify the shower seat and its components in any way.
- Do not use the shower seat if damaged or altered in any way.
- After use, the shower seat should be removed from the tray to avoid obstruction.
- Do not use the shower seat for different purposes from the intended one.
- The shower seat must not be stored outdoors.

MAINTENANCE INFO:

Product maintenance consists of cleaning and sanitation procedures. In order to guarantee safe use and an adequate standard of hygiene, the user should perform these procedures before every use. The user has to make certain of the structural integrity of the device and its components, and the tightness of the screws must be checked. For cleaning and sanitation procedures, follow the steps below:

1. Gloves should be worn.
2. Remove the evident dirt before proceeding with the most accurate disinfection.
3. Use water or non-aggressive detergent for the cleaning procedure.
4. Wipe with a dry and clean cloth.

Alternatively, the product can be cleaned in domestic dishwashers at the maximum temperature. The soft cushion must be cleaned strictly by hand.

REUSE REQUIREMENTS:

The product is suitable for reuse by other users. Before reusing by others, the product must be correctly cleaned and sanitized. Before the transfer of the product to others, the product must be checked by qualified dealers.

CORRECT DISPOSAL PROCEDURE:

Do not dispose of the product in the regular household waste. Ask your local authority beforehand for the correct and environmentally friendly way to dispose of it.

INCIDENT REPORTING:

If there was a serious incident involving this device, contact the manufacturer of this product at info@kingbathrooms.com and the responsible authority in your country.

ITEM CODE	DESCRIPTION	PAGE
KING-ST-KN	Square stool King	44
KING-STA-KN	Square stool height adjustable King	45
KING-STR-KN	Round stool King	44
KING-STRA-KN	Round stool height adjustable King	45
KING-STT-KN	Triangle stool King	44
KING-STTA-KN	Triangle stool height adjustable King	45
KING-STRR-KN	Rotating round stool King	44
KING-STRRR-KN	Rotating round stool height adjustable King	45
KING-STAT-KN	Square stool height adjustable with tray King	46-47
KING-STRAT-KN	Round stool height adjustable with tray King	46-47
KING-STTAT-KN	Triangle stool height adjustable with tray King	46-47
KING-STRRAT-KN	Rotating round stool height adjustable with tray King	46-47
KING-BST-KN	Black square stool King	44
KING-BSTA-KN	Black square stool height adjustable King	45
KING-BSTR-KN	Black round stool King	44
KING-BSTRA-KN	Black round stool height adjustable King	45
KING-BSTT-KN	Black triangle stool King	44
KING-BSTTA-KN	Black triangle stool height adjustable King	45
KING-BSTRR-KN	Black rotating round stool King	44
KING-BSTRRR-KN	Black rotating round stool height adjustable King	45
KING-BSTAT-KN	Black square stool height adjustable with tray King	46-47
KING-BSTRAT-KN	Black round stool height adjustable with tray King	46-47
KING-BSTTAT-KN	Black triangle stool height adjustable with tray King	46-47
KING-BSTRRAT-KN	Black rotating round stool height adjustable with tray King	46-47
KING-CST2/4-KN	Soft cushion for square stool light blue/grey King	48
KING-CSTR2/4-KN	Soft cushion for round stool light blue/grey King	48
KING-CSTRR2/4-KN	Soft cushion for rotating round stool light blue/grey King	48
KING-CSTT2/4-KN	Soft cushion for triangle stool light blue/grey King	48
KING-RD-KN	Rotating disc for stools King	49
KING-BRD-KN	Black rotating disc for stools King	49

ICON	DESCRIPTION
	Made in Italy
	Tools Free Assembly
	Dishwasher Friendly
	Max User Weight
	2 Years Warranty
	Scan QR Code to watch the assembly video

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LM Global Design Ltd.