Rollator BUFFALO HD

Item no. 312305, 312307, 312385 & 312387

Montage- und Gebrauchsanleitung
Montage- og brugervejledning
User Manual
Manuale d'uso e manutenzione

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1. Introduction

Thank you for buying a high-quality Mobilex product. These user instructions contain a detailed product description and important guidelines for correct use of the product. You must read the instructions carefully before using the product. Please always observe all guidelines, in particular the security instructions, in order to use the product safely for a long time.

Intended use of the device

The Mobilex product is medical device which has been designed to increase the mobility of persons who have difficulties standing or walking. The rollator enables those persons to regain the ability to perform daily chores. The rollator is suitable for persons who as a result of various diseases need stable walking support. The rollator should only be used as a support for standing, walking and sitting. Any other use is prohibited.

Indications

The rollator is a medical device, the intended use of which is to assist disabled people when walking. The indications for use are diseases that cause mobility limitations and disabilities, obstructing independent sitting, standing and walking. Therefore, there is a need for an additional stable support when performing the above activity. Indications cannot be related to one specific clinical picture and they depend on individual evaluation of potential user's health status by professional physician or physical therapist, but main target user groups are elderly people, persons with paresis, rheumatic diseases, and other people with limited mobility of the lumbar spine, knees or hips. Using the product for any purpose other than those described above is prohibited.

Contraindications

Rollators are not indicated for those persons who can't maintain proper body balance in a standing position. The device must not be used if the user has problems with impaired perception. Contraindications cannot be related to one specific clinical picture and they depend on individual evaluation of potential user's health status. The choice of orthopedic aid appropriate for a given clinical picture must be always made with the help of a specialist physician or physiotherapist. In addition, observe the safety information in Chapter 2.

If you have any questions, please contact your dealer or Mobilex A/S

Mobilex A/S Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 Email: info@mobilex.dk
--	--

2. Security instructions

- Use only the product if it is technically flawless
- In case of any product damage or faults please contact your dealer immediately
- Observe the data on the product label
- Use the product only for the described purpose
- Please notice that there is a risk of injury at all joints on the rollator
- Always check before use that the wheel brakes function perfectly
- Use the rollator only when it has been fully unfolded and the locking device has been activated
- The rollator should only be used on a flat stable surface, indoors or outdoors
- The seat may only be used when the brakes have been firmly locked
- Both brakes should always be activated or locked simultaneously
- Max. load of the rollator is 200 kg in total
- Max. load of the shopping bag is 5 kg
- It is not allowed to carry an extra person on the seat
- The product must not be changed without the manufacturer's written permission
- The rollator must always be braked when not in use

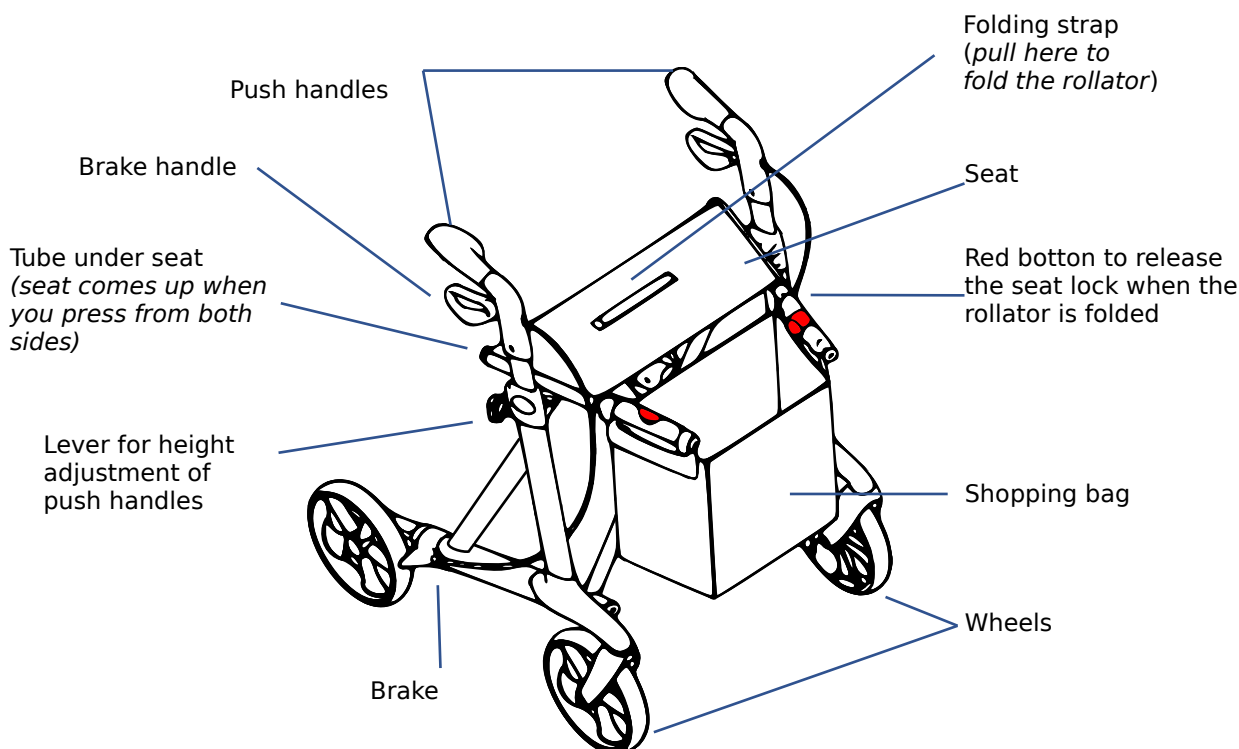
3. Description

The Buffalo rollator has a powder coated reinforced frame with 4 wheels. The light-weight frame makes it easy to fold it together and offers easy storage and transport. The rollator runs easily and is sufficiently stable. The height of the underarm supports can be adjusted. The rollator is delivered with a mounted seat, backbelt, crutch holder and shopping bag.

Parts supplied and mounting instructions

The Buffalo rollator is delivered in a carton with the following contents:

- ✓ This user manual
- ✓ 1 rollator including a mounted seat, back belt, crutch holder and shopping bag

Control upon delivery

After unpacking the rollator please check if anything is missing, if all parts shown in the drawing have been delivered. If any parts are missing, please contact your dealer immediately.

Unfolding the rollator

- Press on the red button on the seat tube lock (see picture on the right)
- Press down both tubes under the seat until the frame unfolds and you can hear a "click". Now the locking device is in a fixed position.
- To be quite sure that the frame has been fully unfolded try to press the rollator together
- Check that the levers for height adjustment have been firmly tightened
- Check that both handles are at the same height and stable

Use of a defective rollator is strictly forbidden

Always check before use the function of all parts of the rollator. If the above recommendations are not followed, injuries may occur.

Folding the rollator

- First detach the strap which is mounted between both push handles
- Pull the strap at the seat until the frame folds together
- Now the rollator is folded together

Height adjustment of underarm supports

- First unlock the strap which is mounted between both push handles
- Loosen the lever by turning it to the left
- Adjust the height
- Turn the lever to the right
- Attach the strap again

Both handles should always be at the same height.

The correct height of the handles should always be the elbow height of the user when he stands with his arms hanging loose.

Length adjustment of push handles

- First detach the strap which is mounted between both push handles
- Loosen the lever by turning it to the left
- Adjust the length
- Turn the lever to the right
- Attach the strap again

Brake function

- To brake you must pull both brake handles towards yourself
- The more you pull the higher the braking capacity
- To ensure that the rollator remains STOPPED you must keep the brake handles pulled
- To make the rollator MOVE you must let go of the brake handles whereby the braking will stop

*Never release the brakes when seated on the rollator!
Always activate the brakes when not using the rollator!
When placing the rollator on a non-plane surface you must always activate the brakes!*

Activating and deactivating brakes

- To stop the rollator pull the brake handles upwards
- To park the rollator push the brake handles downwards until a click is heard
- Do not sit on the rollator unless the brakes are in a locked position
- Before setting the rollator into motion you must stand up in the walking direction and release the brake handles by pulling them upwards

The brakes must always be in a locked position when the rollator is parked!

Never use the seat if the brakes are not locked!

When seated, never lean backwards. It is recommended to sit only at the front part of the seat. Do not sit down on the bag.

Function description

As the frame can be clapped together the rollator can fold, and as a result it requires very little space. Push handles can be freely adjusted to fit the size of your body. The seat allows you to rest.

Accessories

For the Buffalo rollator various accessories are available with the following article no. 312340, 312316, 312320, 312322, 312331, 312349, 312366, 312367, 312368, 31236A, 31236B, 312321. Please contact your dealer.

Transport recommendations

It is recommended to transport the rollator in the folded position. Its construction allows easy unfolding and folding. You should check that the rollator is in a stable position during transport to avoid any deformation.

Use


Stand in the middle behind the back wheels. Grab the push-handles and smoothly start walking. Due to the special design of the brakes, only a minor effort is required to use the brakes.

4. Technical data

Rollator Buffalo HD	Standard Item nr. 312305	Small Item nr. 312307	Standard with SOFT wheels Item nr. 312385	Small with SOFT wheels Item nr. 312387
Height of the push handles (adjustable)	74 - 102 cm	66 - 86 cm	74 - 102 cm	66 - 86 cm
Recommended user heights	150 - 200 cm	125 - 170 cm	150 - 200 cm	125 - 170 cm
Seat height	62 cm	55 cm	62 cm	55 cm
Total width opened / folded	68 / 22,5 cm	68 / 22,5 cm	68 / 22,5 cm	68 / 22,5 cm
Inner width between push handles	56 cm	56 cm	56 cm	56 cm
Length (depth)	66 cm	66 cm	66 cm	66 cm
Weight (+ bag 380 g)	7,8 kg	7,5 kg	7,9 kg	7,6 kg
Max. load (rollator / bag)	200 kg / 5 kg	200 kg / 5 kg	200 kg / 5 kg	200 kg / 5 kg
Brakes	2 lockable with parking function			

Product label

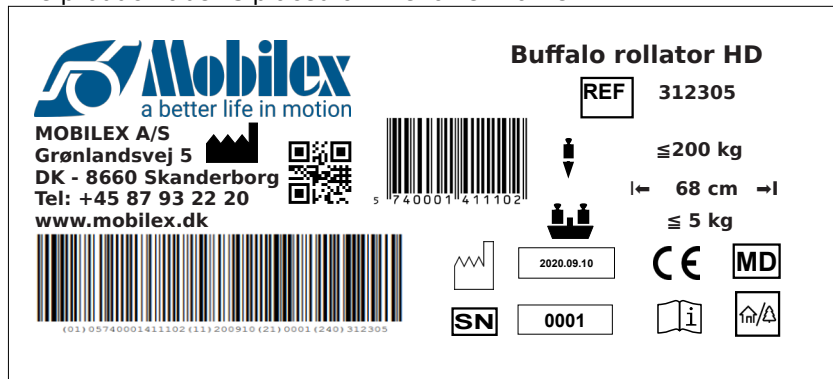The product label is placed on the lower frame

Example of product label Buffalo 312305

	QR-code link to homepage		Manufacturing date
	Manufacturer		Max load
	Observe the user manual		Max width
	To be used indoor and outdoor		serial no
	CE marked product		Item Number
	Max load basket		Medical Device
	Only to be used indoor		

Overview icon meaning

5. Maintenance and care

Normal dirt can be removed with standard cleaning agents and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning and disinfection (do not use alcohol-containing products and no solvents or abrasives).

The device basically does not need maintenance. Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following to be checked:

- ✓ Cleaning and disinfection of the product
- ✓ Frame/product should be checked if it has been bent, damaged, worn or corroded
- ✓ Wear parts are checked and replaced if necessary
- ✓ General retightening of the product

Recommended service life: Normally 10 years depending on use.

Cleaning the wheels

It is recommended to clean the wheels in hot water with a mild cleaning agent. Lubricate the bearings with oil periodically or when the wheels start squeaking.

Warning

 If the product is exposed to very calcareous water, a binding and accumulation of lime on wheel bearings and other moving parts may occur. In this case, the product can be cleaned with a limescale remover and then lubricated with lubricating oil.

Warning

 When used in highly chlorinated environments extra strong corrosion may occur and the warranty will be void.

Warning

 Please note that if the frame is bent or broken, the product must not be recycled but must be replaced

Storage

We recommend storage in a dry place.

Transport recommendations

The device should be transported in a stabilized and secured way with no risk of sudden or dangerous movements during transport.

Reuse

The product is suitable for reuse. Before reusing, where a new user is to take over the product, the product must be checked according to point 5.

6. Troubleshooting

<i>Problems</i>	<i>Possible Causes</i>	<i>Solutions</i>
The rollator is difficult to guide or push	Dirt between wheels and forks	Remove dirt or hair between wheels and forks
The brakes do not work or work only in one side	The brake shoes are worn, or the brakes are not adjusted correctly	Adjust the brakes. Replace the brake shoes if needed (→ service centre)
The seat does not fit in when the rollator is unfolded	The locking device is in a wrong position or defective	Check that the locking device is in a correct position. Replace it if needed (→ service centre)

7. Disposal of the product

The product may not be disposed of with household waste but must be brought to the local recycling site.

8. Guarantee

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

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Hereby declare under our sole responsibility as a legal manufacturer that the product specified on the product list below, meet the essential health and safety requirements and is in conformance with the provisions of the Regulation (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL on medical devices.

The product specified on the product list below is "technical aid for the disabled", classified as Class I, medical device. The classification is based on the requirements of Rule 1 of annex VIII, of the Regulation (EU) 2017/745.

The CE marking has been affixed on the product according to Annex V of the Regulation (EU) 2017/745.

PRODUCT LIST

Buffalo Rollator HD

REF / item no.	312305	312307	312385	312387
UDI-DI	5740001411102	5740001411126	5740001411331	5740001411348
BASIC-UDI-DI	57400014BUFFALOB7			

ACCESSORIES LIST

Item nr.	Accessories item nr.
312305	312340, 312316, 312320, 312322, 312331, 312349, 312366, 312367, 312368, 31236A, 31236B, 312321, 312337, 312338, 31233J, 31233I, 312329
312307	312340, 312316, 312320, 312322, 312331, 312349, 312366, 312367, 312368, 31236A, 31236B, 312321, 312337, 312338, 31233J, 31233I, 31232J
312385	312340, 312316, 312320, 312322, 312331, 312349, 312366, 312367, 312368, 31236A, 31236B, 312321
312387	312340, 312316, 312320, 312322, 312331, 312349, 312366, 312367, 312368, 31236A, 31236B, 312321, 31232J

Harmonized norms used during conformity estimation:

PN-EN ISO 11199-2:2008; EN ISO 14971:2012, EN 1041:2009

In case of unintended incidents or suspected faults, failures or defects of the product, the dealer or Mobilex A/S must be contacted

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Mobilex A/S, Grønlandsvej 5, DK-8660 Skanderborg, Denmark, **SRN: DK-MF-000021885**