DE EN

GEBRAUCHSANWEISUNG: ROLLSTÜHLE GEMINI, 24-ZOLL-HINTERRÄDER
USER MANUAL: WHEELCHAIRS GEMINI, 24" REAR WHEELS

REF : 0811300, 0811301, 08011306, 0811300B, 0811301B,
0811607, 0811609, 0811610, 0811611

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T: +30 28210 63222 | WEB: www.mobiakcare.com

INTENDED USE:

These wheelchairs have been designed for physically or mentally handicapped people with limited mobility. It makes possible to transport people with mobility problems or limited mobility and can be used for movement by the patient himself (if it has 20-24" rear wheels) or with the help of an attendant.

When the wheelchair has a rear wheel with a diameter of 50 to 60 cm and hand rim on the wheel then the user can move independently

SPECIFICATIONS: Please refer on **ANNEX A**

DIMENSIONS: Please refer on **ANNEX B**

ASSEMBLY: Please refer on **ANNEX C**

CRASH TEST: Please refer on **ANNEX D**

SYMBOLS: Please refer on **ANNEX E**

GENERAL WARNINGS:

- If you don't read this user manual, it is preferable not to use this product or another available part. If you don't understand the usage or the precautions, please contact with the dealer or the proper technical person before you use the product because may be caused damage.
- Keep away the product from heat sources.
- Be careful when children are nearby and do not allow the children to play with the product.
- Do not exceed the maximum payload.
- Special care is always required where there are moving parts that can cause limb entrapment and injury.
- Do not attempt to lift the wheelchair by any removable parts.
- Sitting down in and Standing up from the wheelchair can be performed independently but requires strength and mobility on the upper body.

IMPORTANT!

The construction of this wheelchair is designed to provide maximum safety. However, users may be at risk by using their wheelchairs unsuitable. For your own safety, the following rules must be observed. Nonspecific or incorrect adjustment work can increase the risk of accidents.

- Always wear light-colored clothes when moving in dark places so that you can be reached more easily from others.
- Be careful during your first course in this wheelchair.
- Meet your wheelchair.
- Make use of your wheelchair on level ground as well as uphill and downhill.
- Do not attempt to reach objects if you need to lift them from the floor reaching between your knees.
- Do not attempt to reach objects if you need to move on to the seat.
- Do not use on roads or motorways.
- Do not use the wheelchair on a road or surface with water, ice or lubricants.
- Do not use unauthorized parts, accessories, or adapters other than those approved by MOBIAK.
- Do not stand in the wheelchair.

BEFORE USE:

1. Check if the frame is damaged in order to guarantee a safe use of the product. (There are no cracks or fractures in the frame)
2. Check, if the product is properly assembled.
3. Check If the seat is completely open. If not push the seat down from the lateral frame.
4. Check that the brake works correctly.
5. Always check the condition of wear for the mechanical parts to ensure the absolute safe use of the product for people and objects.
6. If there are quick release axes, check that they have been secured correctly.
7. Check tires, tire pressures (if need id) and wheel locks (brakes). The brakes will not work properly if the pressure is too low or the gap between the tire and the wheel locks is too long.

WARNING!

The following methods must be used only from experienced patients. If you are not experienced, ask for help from an attendant.

SITTING DOWN TO THE WHEELCHAIR:

1. Position the wheelchair as close as possible to you.
2. Engage wheel locks.
3. Lean on the frame with arms and slowly go down towards the seat.

STANDING UP FROM THE WHEELCHAIR:

1. Position the wheelchair as close as possible alongside the seat to which you are transferring.
2. Engage wheel locks.
3. Must rely on both handles equally. Supporting only one hand (and unattended) can cause injury.
4. Shift body weight into the seat that you are going to transfer.

OTHER INSTRUCTIONS:

1. Always use lifts and ramps. If these are not available, you can reach your destination with the help of two escorts. They must lift the wheelchair by holding stable parts of the frame. A wheelchair should never be lifted while a passenger is seated. It should be pushed only.
2. Avoid stairs or obstacles, otherwise ask help from your attendant:
3. This wheelchair must be used solely to carry a person at a time on the seat.
4. Users should not sit in wheelchair while are using any means of transport.
5. The wheelchairs have a brake on the back wheels, which should be secured when the wheelchair is not moving.
6. DO NOT push on the footrests to sit or stand up, DANGER OF OVERTHROW. The footrests are used by the user when He or She is seated.
7. DO NOT touch moving parts while moving DANGER OF INJURY

MAINTENANCE:

Periodic review of the procedures below will help to maintain the life and effectiveness of your wheelchair.

WEEKLY

1. Check the wheel pressure by following the instructions on the wheels. If there are pneumatic wheels on the wheelchair.
2. Check that the central bolt of the transverse arm is tight and in good condition.
3. Clean the fabric with soapy water or sufficient products.
4. Clean the metal parts of the chair with a dry cloth or with special products for chrome surfaces or painted surfaces.
5. Check the wheel bolts and adjust if necessary.
6. Check for damage to front and rear wheels.

QUARTERLY

1. Check and adjust if necessary: wheel forks, the wheel axels and the hand-rim of the rear wheels.
2. Grease or oil all components that are subject to friction or impending wear.
3. Make sure that all nuts and bolts are tight, but not too tight.
4. General revision of the chair's status and repair to whatever is necessary.

ANNUALLY

1. We recommend a full review once a year to a qualified service provider.
2. For the safety of the patient, we recommend that the wheelchair will be checked by the manufacturer every 2 years. In case of repair, use only authentic spare parts.

CLEANING:

In addition to maintenance and inspection, both the frame and the upholstery can be thoroughly cleaned.

- To maintain its appearance, the fabric must be cleaned regularly.
- Dirt can be removed with a sponge and a mild detergent with warm water, sweeping in the direction of texture.
- Rinse with clean water before letting it dry.
- **DO NOT leave the wheelchair wet.**
- Do not place the fabric in a tumble dryer. If the dirt is deep, the surface should be cleaned with a soft brush.
- The frame should be cleaned regularly with a mild detergent.
- The wheelchairs with leather seats must be cleaned with dry cloth.
- Clean the wheels with warm water and mild soap. Pay attention in drying the wheels. it is dangerous use the wheelchair with wet wheels.
- If you need to disinfect the product, use a common, mild disinfectant.
- Protect the item from scratches, cuts and punctures.

CAUTION (!):

- In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.
- Do not use the product for a purpose not indicated in this manual.
- MOBIAC S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- MOBIAC S.A reserves the right to change the information contained in this document without previous notice.
- There is a $\pm 3\%$. Deviation for the Dimensions of the Products.

ENVIRONMENTAL PROTECTION

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.

INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

DECLARATION OF CONFORMITY:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

WARRANTY:

The product is guaranteed for a period of two (2) years from the purchasing date.

The warranty covers factory defects & DOES NOT cover damages resulting from misuse, poor maintenance, modification, overuse or non-compliance with the product's operating instructions. Also DOES NOT cover the parts that can wear out during use or over time, like the upholstery, wheels or brakes.

Damages or defects caused by natural disasters, unauthorized maintenance or repair, supply problems (where applicable), carriers are not covered by the warranty

No service or spare part is covered unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the user- customer or by dealer.

WARRANTY CLAIM FORM

BUYERS DETAILS:			
FULL NAME:			
PURCHASING DATE:		PHONE::	
SERIAL NUMBER:			
LOT :			
DEALER DETAILS			
FULL NAME:			
ADDRESS:			
PHONE:			
			SIGN AND SIGNATURE

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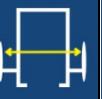
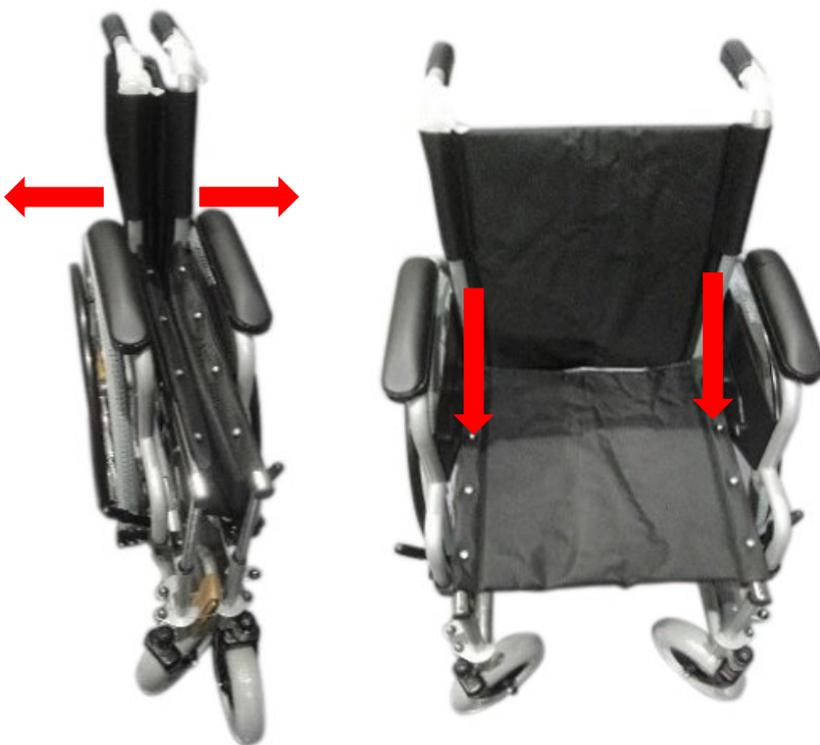
The wheelchair that you have in your hands may differ from the one in the following photos. The assembly process is similar. First, we check the contents of the package.

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To open: Get the handles and pull outwards, then push the bars on the side of the seat, downwards until the wheelchair is completely open.

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To close: Follow the reverse procedure.

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Assembly the Footrest by installing the hinge plates that are on the part onto the hinge pins on the wheelchair frame. Push the footrest towards inside of the wheelchair until it locks into place. Repeat the same procedure for the other footrest.

Remove the Footrest by pushing the release lever, rotate outwards, lift the footrest off the hinge pins.

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Optional with the purchase of the elevating footrests 0811358 & 0811359.

For the ref: 0811607 the Elevating Footrests are standard.

You will not need the footrest belt. Remove the standard footrests and install the elevating footrests with the same way as the standard one.

To move the footrest upwards, pull the main part and leave it at the desired height. To move the footrest downwards, push the lever and lower it, holding it from the lower part. This function, for security reasons, is recommended to be done only from the attendant and not by the user himself.

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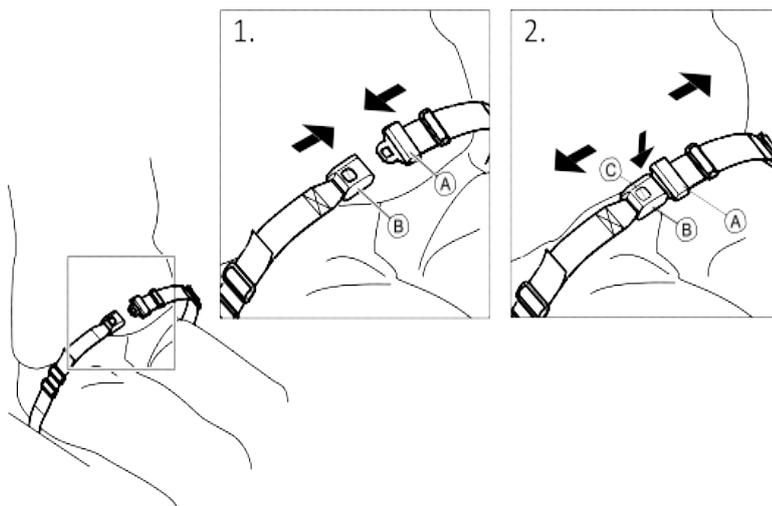
It prevents the user from sliding downward in the wheelchair or from falling out of the wheelchair.

A loose belt can allow the user to slip down and create a risk of strangulation.

Ensure that you are sitting fully back in the seat and that the pelvis is as upright and symmetrical as possible.

To **close**, push the catch A into the buckle clasp B.

To **open**, push the Press button C and pull the catch A out of the buckle clasp B.

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Anti-tilt system (Upon Request- It is not included in the packaging)

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In the case where the wheelchair includes an overturning system, remove the cap from the pipes at the bottom of the wheelchair.

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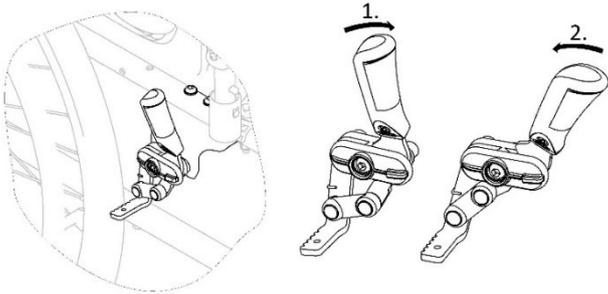
Install the lock pin to the intersecting holes. By inverting the process, we can remove the rollover avoidance system.

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Always remove the anti-tilt wheels before driving over a step or kerb. Never step on the anti-tilt or use it as a tipping aid.

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The brakes are used to immobilize the wheelchair when it is stationary to prevent it from rolling away.

To engage the brake, push the brake lever forwards with the palm. (1)

To disengage the brake, pull the brake lever backwards. (2)

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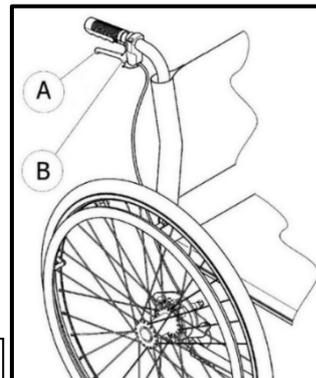
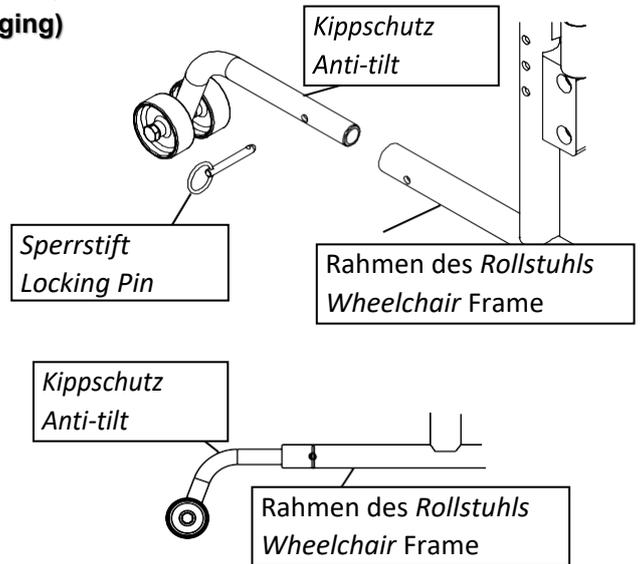
Never engage the parking brakes while you are moving. Do not engage the parking brakes to slow down the wheelchair.

Do not lean on the parking brakes for support or transfer. Keep your fingers away from movable parts when using the brake, always keep your hand on the brake lever.

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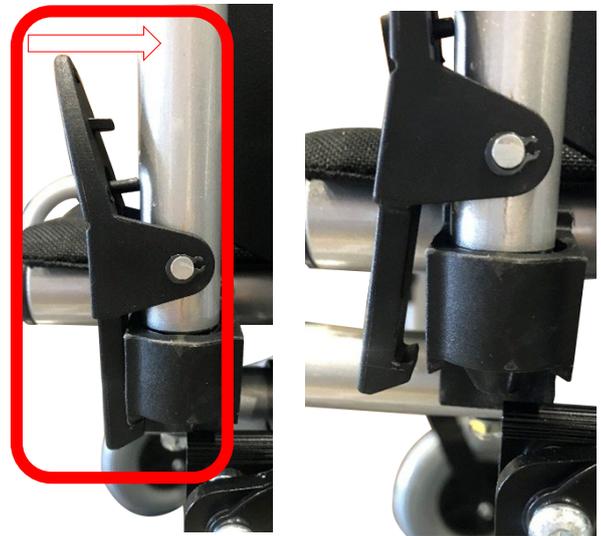
For the Wheelchairs 0811306, 0811301, 0811300, 0811610, 0811611 are standard. For the wheelchairs 0811301B, 0811300B & 0811609, are optional with the extra ref. code 0811398.

*With **attendant's brakes**, an assistant can brake the wheelchair while it is moving. The attendant's brake can also be used as a parking brake.*

To brake, pull the brake lever A on the push handle.
To park, pull the brake lever firmly until the safety lever B engages.
To release, pull the brake lever and release the safety lever underneath the brake lever

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For the Wheelchairs 0811306, 0811301, 0811300, 0811610, 0811611 & 0811607

How to remove the Quick Release rear wheels:

The Wheelchair has a quick-release system on the rear wheels so they can be removed easy and can reduce also the storage volume in the car and home.

To disassemble the wheel, press the black button in the center of the wheel, to unlock, and at the same time pull the wheel outwards until it comes out completely from its base.

Repeat the same process on the other wheel.

To assemble follow the reverse procedure.

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*Our wheelchair **0811300** has successfully passed the crash test according to ISO 7176-19:2022. This means that the wheelchair is safe to be pushed inside a suitable vehicle and secured during travel and has anchor points, allowing the wheelchair to be secured into place with the occupant seated within the wheelchair. The anchor points on the wheelchair are used by attaching straps to the anchor points on the wheelchair. These straps are then tightened to ensure the wheelchair will not move in the event of an accident and therefore protecting the wheelchair user from harm when travelling.*

(Rollstuhl 0811300 während des Crash-Tests)
(Wheelchair 0811300 during crash test)

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In accordance with ISO 7176-19 if the user is seated in the wheelchair this:

- 1) *Must be located in a forward facing position, in the direction of travel (see below picture 1), not facing sideways and must be secured using the wheelchair tie-down system (**these systems must meet the requirements of ISO 10542**).*

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2) *Only the positions marked with the symbol for the attachment point should be used (see picture 2). These can usually be found at the rear left and right and at the front left and right as shown in the below picture 3.*

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3) *The wheelchair brakes must be applied.*

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4) *The wheelchair must be fixed with a 4-point wheelchair restraint system anchored to the vehicle in accordance with ISO 10542.*

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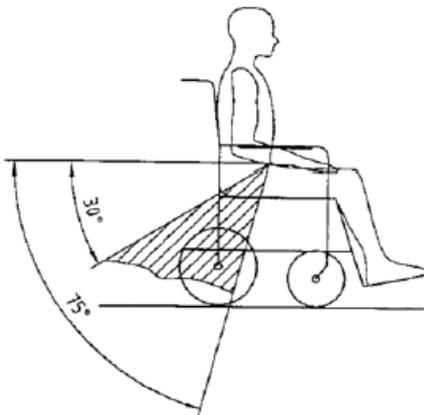
Wear the pelvic belt low across the front of the pelvis, so that the angle of the pelvic belt is within the preferred zone of 30° to 75° to the horizontal, similar to that shown below.

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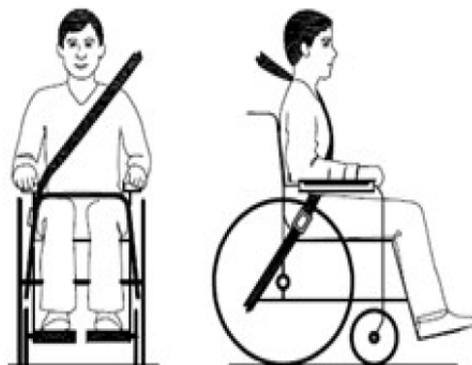
Restraint belts must not be placed over wheelchair components such as armrests or wheels, so that cannot be held away from the body. They should be placed over the shoulder and diagonally across the chest (see below pictures).

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BELT RESTRAINTS MUST NOT BE HELD AWAY FROM BODY BY WHEELCHAIR COMPONENTS SUCH AS ARMRESTS OR WHEELS

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BELT RESTRAINTS SHOULD MAKE FULL CONTACT WITH THE SHOULDER, CHEST AND PELVIS AND PELVIC BELTS SHOULD BE POSITIONED LOW ON THE PELVIS NEAR THE THIGH-ABDOMINAL JUCTION

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Description of symbols that have been printed on the label, Master Carton and or User Manual

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