WALL MOUNTED FOLDING SUPPORT RAIL

Art. 311655, 311676

Gebrauchsanleitung
Brugervejledning
User Manual
Manuale
Bruksanvisning
Instrukcja Obsługi
Manuel de l'utilisateur

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mobilex.dk/medias/2022-04/klapbar_toiletstoette_311655_76_10044.pdf

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1. Introduction

Thank you for buying a high-quality Mobilex product. These instructions of use contain a description and important information about the use of the medical device. It is intended to help you to use the product correctly and safely. It is very important to read the instructions carefully before using the device. Pay particular attention to the safety guidelines and always follow them. Mobilex is constantly trying to improve and upgrade its products. Therefore we reserve the right to change specifications and technical data of the product without further notice.

Intended use of the device

The Wall Mounted folding support rail is a medical device intended only to be used by persons who need support when sitting down or when standing up while using the toilet. The device is suitable for persons who as a result of various diseases need a stable support to stand up and/or sit down. The device should be used inside bathrooms only.

Indications

The device is indicated for people with functional limitations, people in wheelchairs and other people with limited mobility of the lumbar spine, knees or hips, and for all those who have difficulties performing everyday activities on the toilet. Any use other than stated above is prohibited.

Contraindications

The device is not indicated for those patients who can't maintain proper body balance in a sitting and standing position. The device must not be used if the user has impaired perception. Contraindications cannot be related to one specific clinical picture, and they depend on individual evaluation of potential user's health status. The choice of orthopedic aid appropriate for a given clinical picture must be always made with the help of a specialist physician or physiotherapist. In addition, observe the safety information in Chapter 2.

Areas of use

The product can be used in home care, health and long-term care facilities, and hospitals.

Medical incidents and unwanted events

In case of any unwanted events related to the device, please contact the dealer or manufacturer:

Mobilex A/S Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 Email: info@mobilex.dk
---	--

2. Safety requirements

- Make sure that this manual has been read by all persons using the device.
- The manufacturer assumes no responsibility for damage and/or injury resulting from failure to follow the operating instructions.
- Only use the product in a flawless condition.
- If defects or errors are detected you must immediately stop using the device and contact the dealer.
- Follow all manufacturer's recommendations and warnings.
- Be aware of the information on the product label.
- Only use the product for the described purpose.
- Don't climb onto the device.
- Avoid making constructive changes to the device, unless you have the manufacturer's written acceptance for such modifications.
- The device must not be loaded with more weight than indicated in Technical data.
- Use caution when assembling or adjusting any parts of the device.
- Always check stability of the device before use.
- Before use, the product should be completely assembled and well attached to the wall. Please contact us if there are any problems in this regard.
- Folding and adjustable mechanisms can become a cause of danger if body parts get into the gap between components and could become trapped when the gap narrows.
- Always check that the device is securely fastened before each use.
- Use caution on wet or slippery surfaces.
- Clean the product regularly.

Parts supplied

The device is supplied in a carton box. The contents of the package are:

- ✓ 1 Foldable handle;
- ✓ 1 User manual;

Preliminary check

Check the contents of the box with the list above. If at this stage you discover that a part is missing or damaged, contact your supplier immediately.

3. Description

Product description

The wall mounted folding support rail is made of durable steel, powder coated with a special white enamel. It consists of a flat bracket, used for mounting, and a bent tube, which is the handrail. The diameter of the tube and its shape make it easy and safe to use. It has a hinged mechanism that allows it to be raised to an upright position, so it doesn't take up much space when not in use.

Assembly description

Tools needed for installation: electric drill, level, wrench or screwdriver. Additional necessary materials: 6 screws along with wall plugs, of the type and size suitable for the type of wall.

1. Using a level, determine the position of the handrail on the wall, at a height selected for the height of the person using this aid and in a position suitable for the functionality in the surrounding area. The handrails should be fixed so that the movement of tilting the handrail is vertical and upward.
2. Mark the attachment points and drill 6 holes with a diameter suitable for the expansion bolts suitable for proper installation of the handrail. The expansion bolts should be selected according to the type of substrate (e.g. brick, hollow brick, hollow block, gypsum board, cement bonded tiles).
3. Insert the expansion bolts into the wall. Place the handrail bracket on the holes in the wall, making sure that they are exactly aligned with the handrail bracket. Put the screws through the holes of the handrail bracket and screw them into the expansion bolts.
4. Tighten the screws with a suitable tool and check that the handrail is securely fastened to the wall.

NOTE: The selection of materials, method and quality of installation is the responsibility of the person carrying out this installation!

NOTE: Make sure that the chosen wall is suitable to support the handrail and the weight of the user.

NOTE: Test load the handrail to make sure it is securely attached to the wall before you start using it.

NOTE: The installation of the handrail should be carried out by qualified personnel.

Function description

Folding the handrail

To use the handrail, lower it down to a horizontal position, and when out of use, lift it back up to the vertical position.

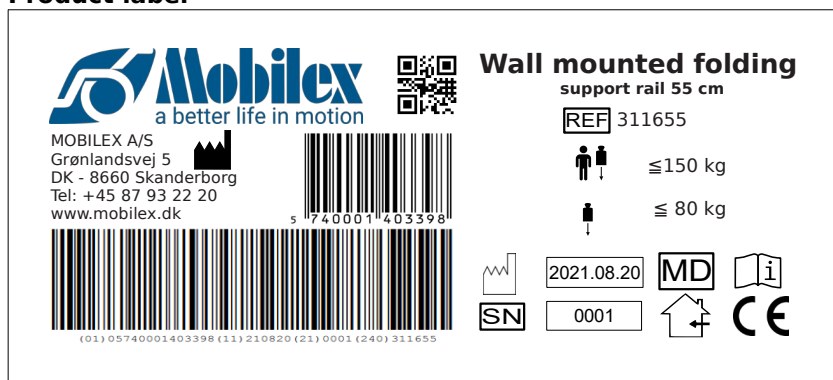
Accessories

There are no accessories provided for this device.

4. Technical data

Wall mounted folding support rail	Art. 311655	Art. 311676
Length	55 cm	76 cm
Weight	2,1 kg	2,5 kg
Diameter	3,2 cm	
Max. user weight	150 kg	
Max. load	80 kg	
Material	Powder coated steel	

Product label

Example of the product label

	QR-code link to homepage		CE marked product
	Manufacturer		Manufacturing date
	Observe the user manual		Max load
	Serial no.		Medical Device
	Item Number		Max user weight
	Only to be used indoor		

Meaning of the icons

5. Care and maintenance

Regular dirt can be removed with standard cleaning products and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning. The entire product can be disinfected with a commercially available disinfectant. Always follow the disinfectant manufacturer's instructions for use and safety when disinfecting.

Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following:

- ✓ Cleaning and disinfection of the product
- ✓ Checking the frame for bends, damage, wear or corrosion
- ✓ Checking wear parts and replacing them if necessary
- ✓ General re-tightening of the components

Recommended service life: Normally 5 years, depending on use.

Storage: We recommend storage in a dry place in temperatures above 0 °C.

Transportation

The device is not designed for transportation as it is permanently fixed to the wall.

Reuse

The product is suitable for reuse. Before a new user takes over the product, the device must be checked on a technical level and disinfected according to point 5.

6. Most common problems and solutions

<i>Problems</i>	<i>Possible causes</i>	<i>Solutions</i>
The handrail doesn't move smoothly	The hinge is dirty	Clean the hinge

7. Disposal of the product

The product can not be disposed of with household waste and must be brought to the local recycling center.

8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

9. Declaration of conformity

The device has passed the conformity assessment in accordance with EU Regulation: MDR 2017/745 regarding medical devices and is CE marked. A declaration of conformity has been issued for the device and is available at:

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